R2120 - R2121
R2140 - R2141
R2142 - R2143

R4820 - R4825
R4826 - R4840
R4842

MATRIX wash basin lifter – Operation and maintenance manual en

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Content

Symbols used in this manual	4
General safety	5
Product label	6
Intended use	7
Intended operator profile	7
Operating environment	7
Assembly and installation	8
Versions	9
Compatible wash basins	10
Operating instructions	11
Cleaning	12
Maintenance	13
Maintenance inspection	13
Dimensions (mm)	14
Dimensions (inches)	16
Trouble-shooting	18
Technical data	18
Disposal and recycling	18

en

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.


CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

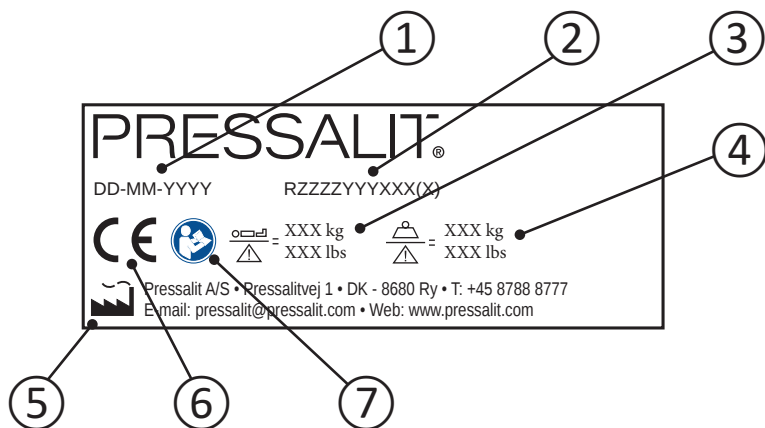① Production date

② Article number

③ Maximum user weight

④ Maximum rated load

⑤ Name and address of manufacturer

⑥ This product is CE-marked in accordance with the European Medical Device Directive 93/42/EEC, including amendments incorporated in Directive 2007/47/EEC. A declaration of conformity can be found at www.pressalitcare.com

⑦ Consult manual for use

Product label placement ①.

Intended use

This product is intended for optimum ergonomics at a wash basin, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room, in private homes, nursing homes and hospitals.

en

CAUTION

Never place objects underneath the wash basin.

Assembly and installation

For assembly and installation, please refer to the assembly instruction supplied with the product.

R4840 - R4842 - R2140
R2141 - R2142 - R2143

PRESSALIT®
care

300

(EN) Mounting instruction

(DA) Monteringsvejledning

(DE) Montageanleitung

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(NL) Montage-voorschrift

(SE) Monteringsanvisning

(NO) Monteringsveiledning

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Subject to alteration without further notice. Ret til ændringer forbeholdes. Recht auf Änderungen wird vorbehalten.
Nous nous réservons le droit d'apporter toutes modifications pour l'amélioration de nos produits sans préavis. Ændringer forbeholdes.
Allein ist Änderungen vorbehalten. Vi forbeholder os selv til ændringer. Salvo alterações.

CE

 keep living

Pressalit Care
Pressalit 1
DK-8860 Ry
Denmark

Telephone
Number
E-mail
Web

+45 87 58 87 77
+45 87 58 86 86
pressalit@pressalit.com
www.pressalit.com

R2120 - R2121

PRESSALIT®
care

300

(EN) Mounting instruction

(DA) Monteringsvejledning

(DE) Montageanleitung

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(NL) Montage-voorschrift

(SE) Monteringsanvisning

(NO) Monteringsveiledning

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Subject to alteration without further notice. Ret til ændringer forbeholdes. Recht auf Änderungen wird vorbehalten.
Nous nous réservons le droit d'apporter toutes modifications pour l'amélioration de nos produits sans préavis. Ændringer forbeholdes.
Allein ist Änderungen vorbehalten. Vi forbeholder os selv til ændringer. Salvo alterações.

CE

 keep living

Pressalit Care
Pressalit 1
DK-8860 Ry
Denmark

Telephone
Number
E-mail
Web

+45 87 58 87 77
+45 87 58 86 86
pressalit@pressalit.com
www.pressalit.com

en

8

Versions

R4820, R4825, R4826

R4840

R4842

en

Compatible wash basins

R2140 (Fits R4840)

R2141 (Fits R4840)

R2142 (Fits R4842)

R2143 (Fits R4842)

R2120 (Fits R4820, 25,26)


R2121 (Fits R4820,25,26)


en

Operating instructions

The wash basin lifter can be adjusted 200 mm up and down.
Max load is 150 kg.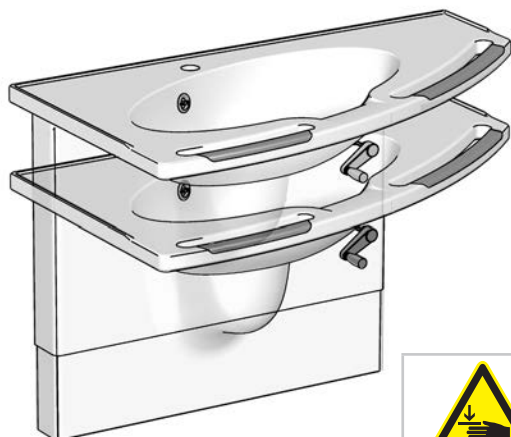

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.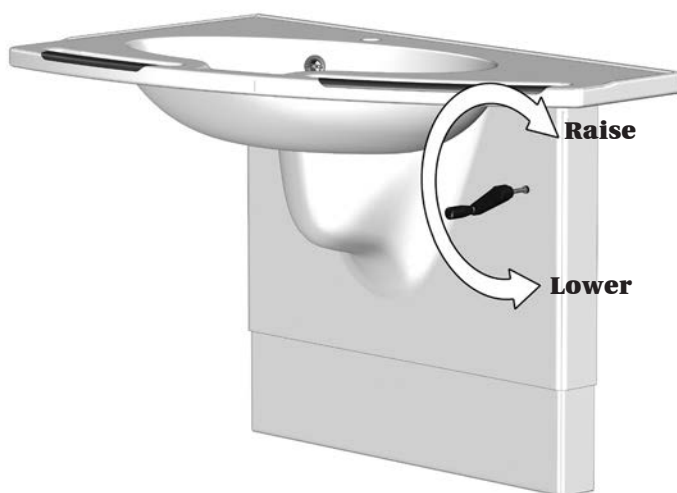

Turn the handle clockwise to raise the wash basin, and counterclockwise to lower it.

Cleaning

Clean the product after each use. Use ordinary soapy water. Rinse off with fresh water and dry with a soft cloth.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids or chloride.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of aggressive, abrasive, corrosive or chlorine-based cleaning or disinfecting products.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

Maintenance



Expected service life of the wash basin lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a monthly basis to adjust the product to its maximum positions up and down to lubricate the internal parts.

Maintenance inspection

Take the wash basin to its highest position.

Remove the screws and the upper front cover ①.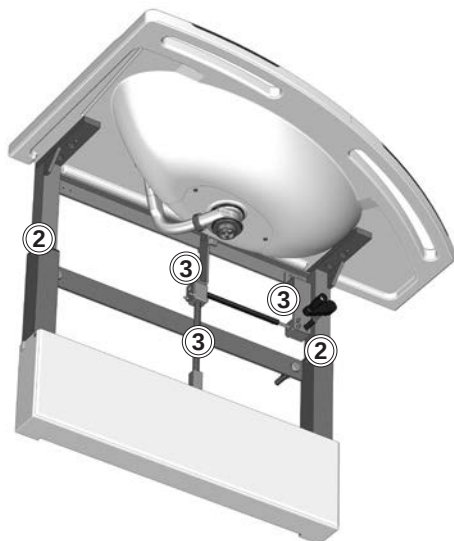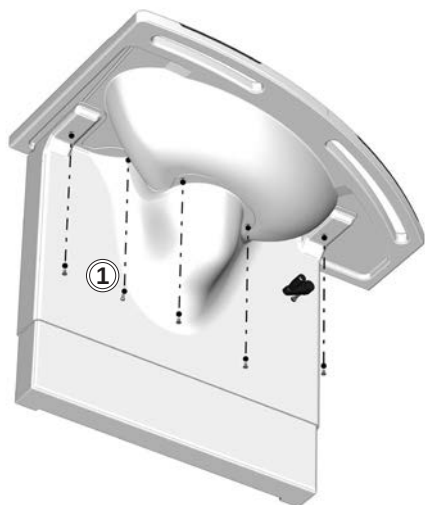


Lubricate the inner square tubes ② and the hoisting mechanism ③ with WD-40 or equivalent.


Take the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

Reattach the front covers.


Dimensions (mm)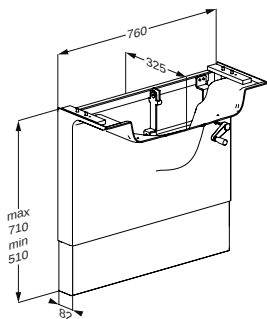


R4820, R4825, R4826

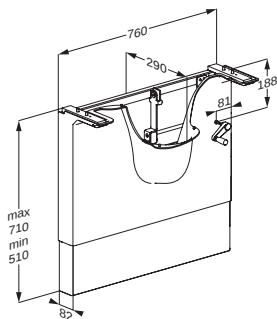
R4840

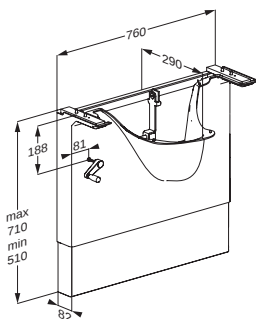
R4842

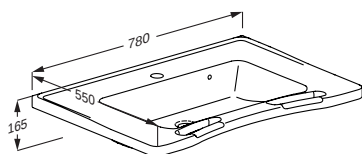
R2120

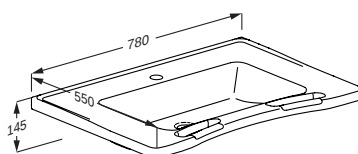
R2121

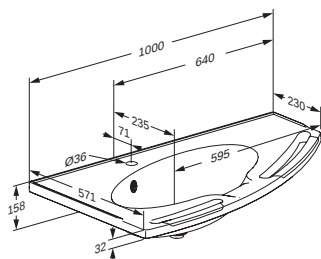
R2140

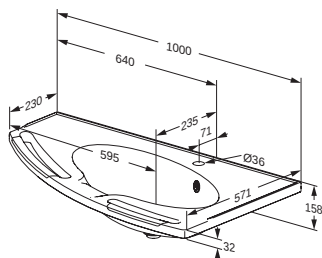R2142

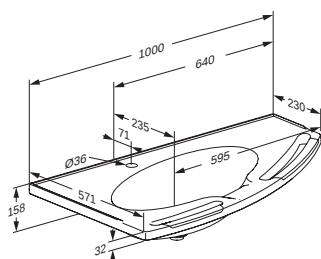
R2141

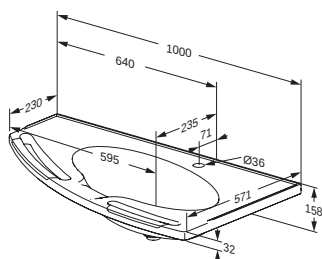
R2143


Dimensions (inches)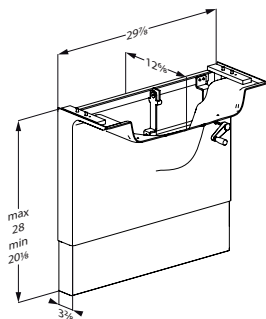


R4820, R4825 and R4826

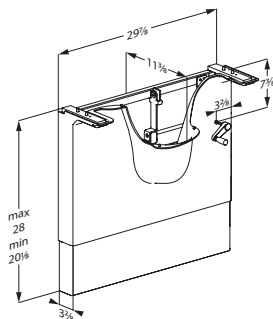
R4840

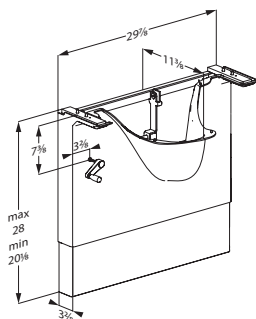
R4842

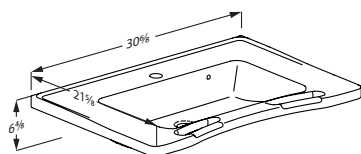
R2120

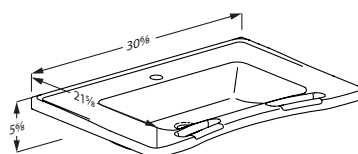
R2121

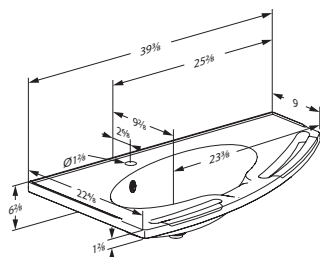
R2140

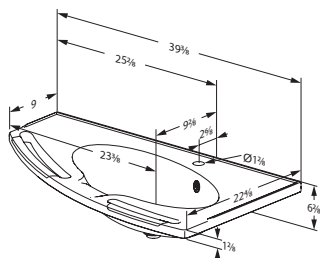R2142

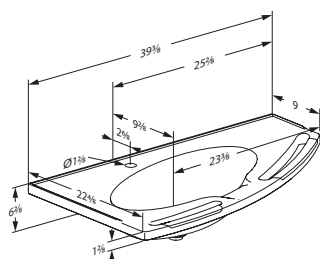R2141

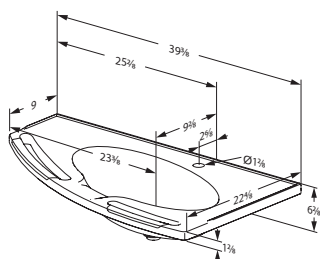R2143

Trouble-shooting

Symptom	Cause	Action
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Technical data

Max load	150 kg / 330 lbs
Materials	Stainless steel Plastics
Weight	18 kg / 40 lbs

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

en

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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Pressalit A/S