R2122 - R2123
R2150 - R2151
R2152 - R2153
R2170 - R2172

R4822 - R4827
R4850 - R4852
R4855 - R4857
R4870 - R4875

MATRIX wash basin lifter – Operation and maintenance manual **en**

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Content

Symbols used in this manual	4
General safety	5
Product label	6
Intended use	8
Intended operator profile	8
Operating environment	8
Assembly and installation	9
Versions	10
Compatible wash basins	11
Operating instructions	12
Safety rail	13
Resetting the wash basin lifter	13
Cleaning	14
Maintenance	15
Maintenance inspection	15
Dimensions	18
Technical data	19
Trouble-shooting	19
Disposal and recycling	20

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Electricity or potentially dangerous voltage.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

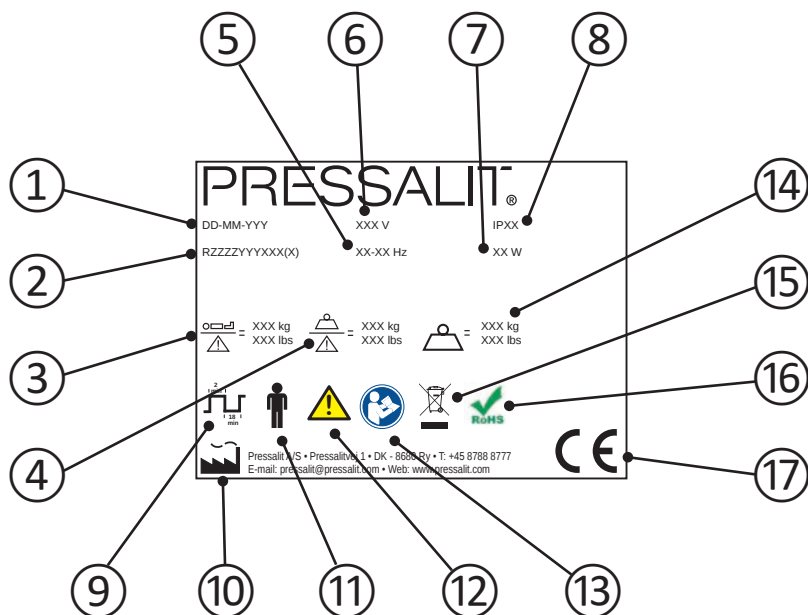
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

en

① Production date

② Article number

③ Maximum user weight

④ Safe working load

⑤ Frequency

⑥ Mains voltage

-
- ⑦ Maximum power consumption
-
- ⑧ IP class
-
- ⑨ Duty cycle The motor on height-adjustable products can be used for a maximum of 2 minutes of continuous operation, followed by 18 minutes pause.
-
- ⑩ Name and address of manufacturer
-
- ⑪ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function The product complies with requirements of IEC 60601-1 to provide protection against electric shock.
-
- ⑫ Consult the manual for use for important cautionary information such as warnings and precautions.
-
- ⑬ Consult manual for use
-
- ⑭ Product weight
-
- ⑮ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
-
- ⑯ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
-
- ⑰ This product is CE-marked in accordance with the European Medical Device Directive 93/42/EEC, including amendments incorporated in Directive 2007/47/EEC. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery and with Council Directive 2011/65/EC (of 8 June 2011) on RoHS. A declaration of conformity can be found at www.pressalitcare.com.

Product label placement ①.

Intended use

This product is intended for optimum ergonomics at a wash basin, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room.

en The product is intended to be used at room temperature, in a stand-alone environment, in private homes, nursing homes and hospitals under normal working conditions.

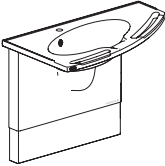
Never place objects underneath the wash basin.

Assembly and installation

For assembly and installation, please refer to the assembly instruction supplied with the product.

R4820 - R4822 - R2150
R2151 - R2152 - R2153

PRESSALIT®
CARE

☞ Mounting instruction

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☞ Montage-voorschrift

☞ Monteringsanvisning

☞ Monteringsveiledning

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Subject to alteration without further notice. Not to be changed without permission. Please refer to the assembly instruction for further information. The assembly instruction is part of the product. Please refer to the assembly instruction for further information. The assembly instruction is part of the product. Please refer to the assembly instruction for further information.

CE

keep living

Pressalit Care

Pressalit Care

DK 8820 Ry

Denmark

Telephone

+45 87 88 27 11

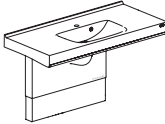
E-mail

pressalit@pressalit.com

www.pressalit.com

R4820 - R4822 - R2170
R2172

PRESSALIT®
CARE

☞ Mounting instruction

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☞ Montage-voorschrift

☞ Monteringsanvisning

☞ Monteringsveiledning

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Subject to alteration without further notice. Not to be changed without permission. Please refer to the assembly instruction for further information. The assembly instruction is part of the product. Please refer to the assembly instruction for further information. The assembly instruction is part of the product. Please refer to the assembly instruction for further information.

CE

keep living

Pressalit Care

Pressalit Care

DK 8820 Ry

Denmark

Telephone

+45 87 88 27 11

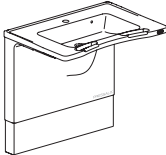
E-mail

pressalit@pressalit.com

www.pressalit.com

R4822 - R4827 - R2122
R2123

PRESSALIT®
CARE

☞ Mounting instruction

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☞ Montage-voorschrift

☞ Monteringsanvisning

☞ Monteringsveiledning

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Subject to alteration without further notice. Not to be changed without permission. Please refer to the assembly instruction for further information. The assembly instruction is part of the product. Please refer to the assembly instruction for further information. The assembly instruction is part of the product. Please refer to the assembly instruction for further information.

CE

keep living

Pressalit Care

Pressalit Care

DK 8820 Ry

Denmark

Telephone

+45 87 88 27 11

E-mail

pressalit@pressalit.com

www.pressalit.com

en

9

Versions

R4822, R4827

R4850, R4855

R4852, R4857

R4870, R4875

Compatible wash basins

R2122 (Fits lifter R4822 and R4827)

R2123 (Fits lifter R4822 and R4827)

R2150 (Fits lifter R4850 and R4855)

R2151 (Fits lifter R4850 and R4855)

R2152 (Fits lifter R4852 and R4857)

R2153 (Fits lifter R4852 and R4857)

en

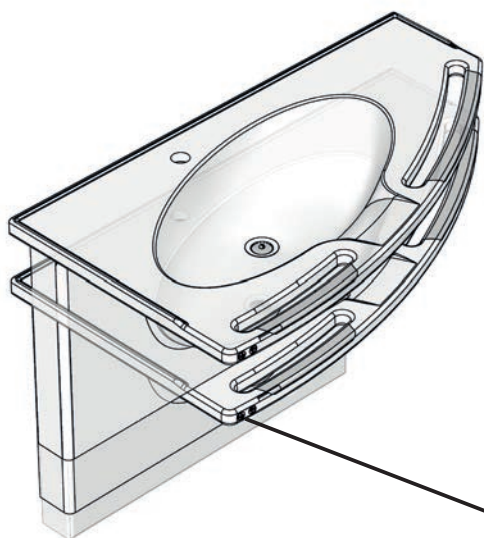
R2170 (Fits lifter R4870 and R4875)

R2172 (Fits lifter R4870 and R4875)

Operating instructions

The wash basin lifter can be adjusted 200 mm up and down.
Max load is 150 kg.

The height adjustment is controlled by operating buttons in the front edge of the wash basin

Press and hold the Up button to raise the wash basin, and press and hold the Down button to lower it. When the button is released, the motion stops.

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

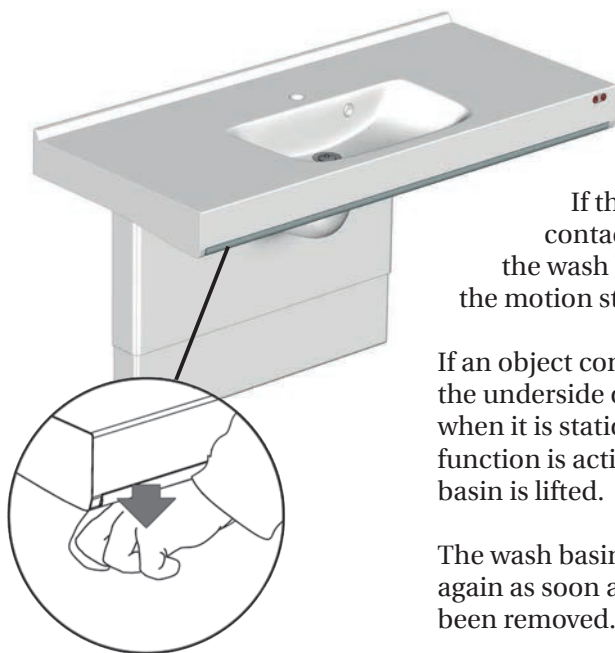
CAUTION

Never place objects underneath the wash basin.

Safety rail

The wash basin lifters **R4870** and **R4875** are equipped with a safety rail.

If the safety rail comes in contact with an object when the wash basin is being lowered, the motion stops immediately.

If an object comes in contact with the underside of the wash basin when it is stationary, the safety function is activated and the wash basin is lifted.

The wash basin lifter can be used again as soon as the obstruction has been removed.

Resetting the wash basin lifter

If the wash basin lifter is jammed, try to reset it.

- ① Make sure that the safety rail is not activated. Lower the wash basin lifter all the way down to its starting position.
- ② Release the lowering button, and after 3-4 sec. press it in again until you hear a slight click after approx. 10 sec.
- ③ Release the lowering button and then raise the wash basin lifter to its highest position. You will hear another click.
- ④ Lower the wash basin lifter a couple of centimeters.
- ⑤ Raise it again to its highest position. You will hear another click.
The wash basin lifter is now reset. The wash basin lifter should also be reset if there has been an interruption of the power supply.

Cleaning

Clean the product after each use. Use ordinary soapy water. Rinse off with fresh water and dry with a soft cloth.

For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids or chloride.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of aggressive, abrasive, corrosive or chlorine-based cleaning or disinfecting products.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

Maintenance

Expected service life of the wash basin lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that maintenance inspection is done once a year.

Maintenance inspection

Take the wash basin to its highest position.

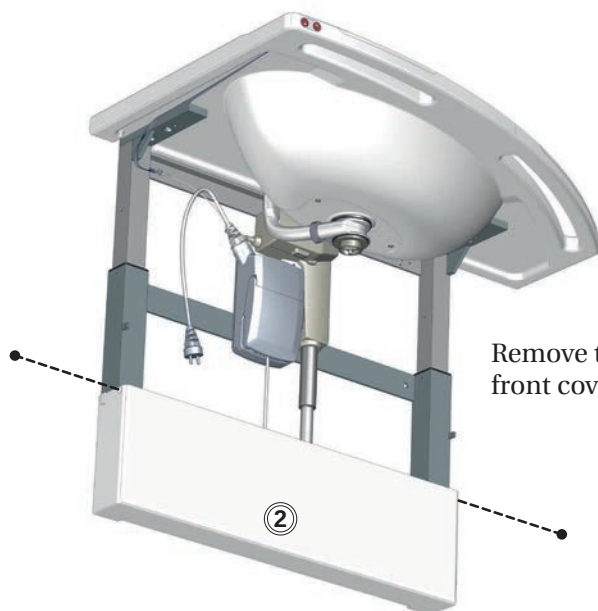
Disconnect mains before removing the covers.


Remove the screws and the upper front cover ①.

For R4822, R4827, R4850, R4855, R4852 and R4857

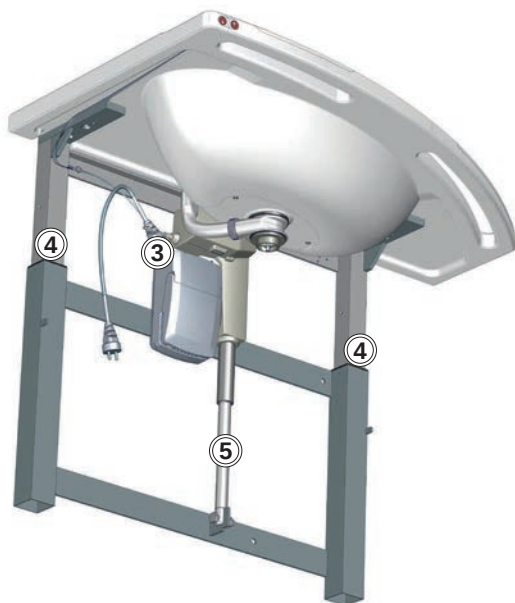Remove the screws and the lower front cover ②.

Check all electrical connections ③.

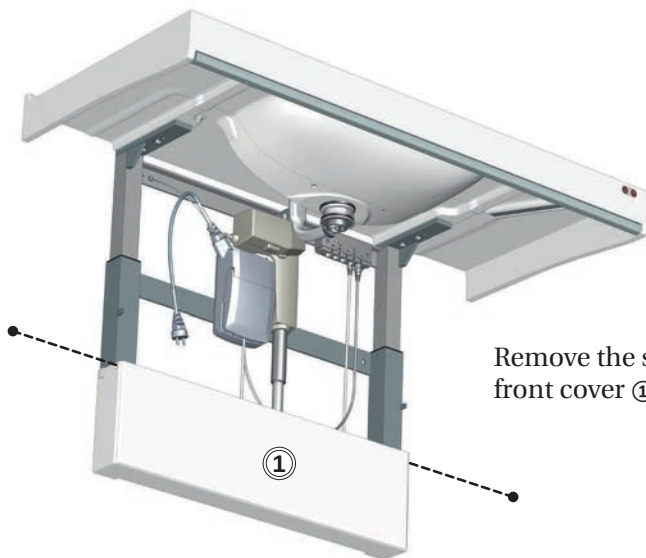
Lubricate the inner square tubes ④ and the actuator rod ⑤ with WD-40 or equivalent.

Reattach the front and the top cover.

Reconnect mains.

Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

For R4870 and 4875


Remove the screws and the lower front cover ①.

Check all electrical connections ②.

Lubricate the inner square tubes ③ and the actuator rod ④ with WD-40 or equivalent.

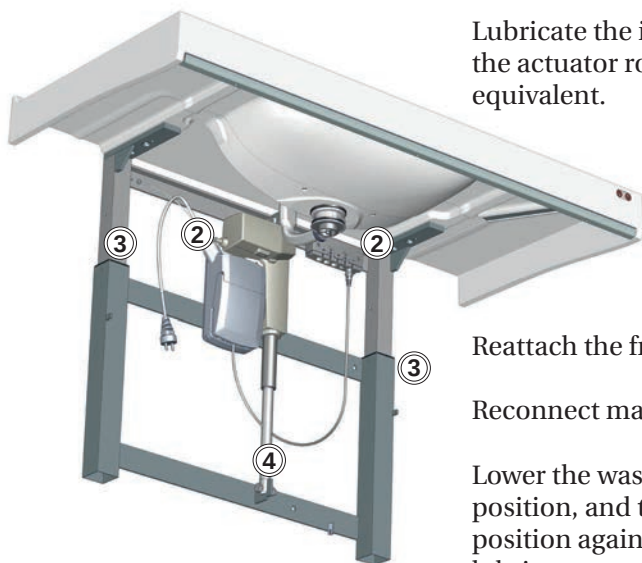en


Reattach the front and the top cover.


Reconnect mains.

Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

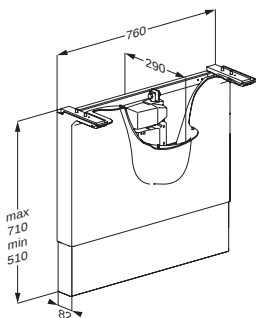
Dimensions


R4822, R4827, R4850
R4852, R4855, R4857

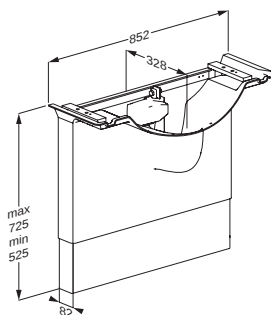
R4870, R4875

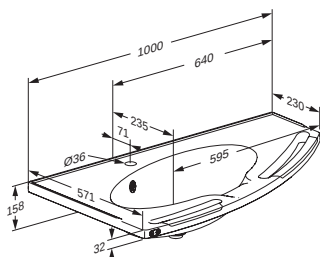
R2150, R2151

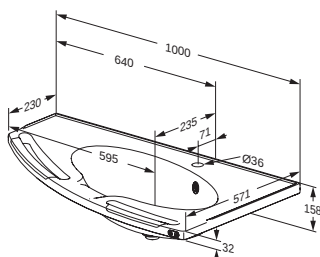
R2152, R2153

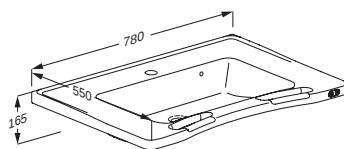R2122, R2123

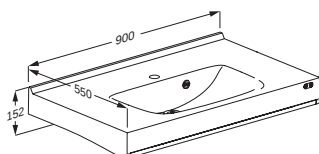R2170

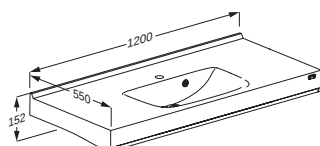R2172

Technical data

Max load	150 kg / 330 lbs
Materials	Stainless steel Plastics
Weight	18,8 kg / 41,44 lbs

Trouble-shooting

Symptom	Cause	Action
Operating buttons does not work.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under "Maintenance".
Safety system does not work.	Faulty switches.	Check whether one or more of the safety system contacts is stuck, and if so, try to repair. If the problem cannot be fixed, replace the contact(s). The safety system will only work when current can be detected. Remember to reset as described under "Resetting the wash basin lifter".

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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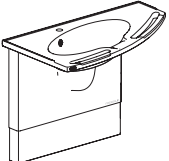
Subject to alteration without further notice. Not to be changed. Subject to change without notice. Not to be changed. Subject to change without notice. Not to be changed. Subject to change without notice. Not to be changed.

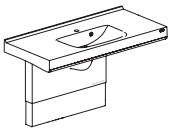
CE

Pressalit Care Telefon: +45 87 88 87 77 E-mail: pressalit@pressalit.com
 Pressalit Care Telefon: +45 87 88 88 88 E-mail: pressalit@pressalit.com
 Pressalit Care Telefon: +45 87 88 88 88 E-mail: pressalit@pressalit.com

**R4870 - R4875 - R2170
R2172**

PRESSALIT®
 CARE

☞ Mounting instruction
 ☞ Monteringsvejledning
 ☞ Montageanleitung
 ☞ Notice de montage

☞ Montage-voorschrift
 ☞ Monteringsanvisning
 ☞ Monteringsveiledning
 ☞ Instrucciones de montaje

Subject to alteration without further notice. Not to be changed. Subject to change without notice. Not to be changed. Subject to change without notice. Not to be changed.

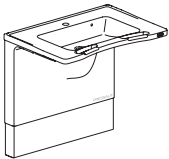
CE

Pressalit Care Telefon: +45 87 88 87 77 E-mail: pressalit@pressalit.com
 Pressalit Care Telefon: +45 87 88 88 88 E-mail: pressalit@pressalit.com
 Pressalit Care Telefon: +45 87 88 88 88 E-mail: pressalit@pressalit.com

**R4822 - R4827 - R2122
R2123**

PRESSALIT®
 CARE

☞ Mounting instruction
 ☞ Monteringsvejledning
 ☞ Montageanleitung
 ☞ Notice de montage

☞ Montage-voorschrift
 ☞ Monteringsanvisning
 ☞ Monteringsveiledning
 ☞ Instrucciones de montaje

Subject to alteration without further notice. Not to be changed. Subject to change without notice. Not to be changed. Subject to change without notice. Not to be changed.

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